



PhEAST – Working Practice Document
**Title: Decontamination of the PhEAST equipment during
the COVID-19 pandemic, No 004**

Please ensure when cleaning and disposing of any waste that national and local trust policy/procedures and guidelines are followed.

❖ Prior to cleaning the Phagenesis equipment

Prior to cleaning the Phagenesis equipment you will need to follow the local trust policy for hand hygiene and wear the specified personal protective equipment.

❖ Cleaning and disinfecting of Phagenesis Equipment

It is a requirement that the Phagenesis base station is cleaned and disinfected after each use.

Universal disinfectant wipes (e.g. Clinell) can be used to clean the base station.

❖ Procedure for cleaning the device and cuff

Please wipe the device in an S-shaped pattern from top to bottom of the device ensure all areas have been covered.

If you need to repeat the process, please use a second wipe and dispose of the first wipe as per local policy guidelines.

Do not use the wipe more than once and ensure of its correct disposal.

The base station can then be left to air dry.

Once cleaned please retain the base station in locked storage.